

Flexitime®

## Flexitime® Fast & Scan – The Perfect Scan. The Perfect Fit.

Providing a way to digitize your traditionally taken impression.

Our scannable A-silicone supports CAD/CAM manufacturing of prosthetic restorations and offers optimal workflow integration for impressions and their digitalization – without the need of powder.

- Flexitime Fast & Scan offers high precision for 1–3 unit impressions.
- The material saves time and increases efficiency by offering a flexible working time of 0:30 to 1:30 minutes and a reduced intraoral setting time of 2:00 minutes for greater patient comfort.
- As a state-of-the-art scannable impression material it is the ideal supplement to cara Scan 4.0i.



### Contact in the USA

Heraeus Kulzer, LLC  
4315 S. Lafayette Blvd.  
South Bend, IN 46614  
Phone: (800) 431-1785  
Fax: (800) 522-1545  
customer.servicehkn@  
kulzer-dental.com



### Contact in Germany

Heraeus Kulzer GmbH  
Grüner Weg 11  
63450 Hanau  
info.dent@kulzer-dental.com

[www.heraeus-kulzer-us.com](http://www.heraeus-kulzer-us.com)

cara®

## cara Scan 4.0i State-of-the-art precision for a perfect fit of indirect restorations.



easi team



## cara Scan 4.0i

Gain convenient and affordable access to automated processing.  
Meet the easi team!

The easi team consists of an

**e**xtra-oral scanner for

**a**utomated processing with


**S**ervice from Kulzer and a scannable

**i**mpression material for high precision.

Thanks to the easi team, we can support your entry into automated processing and ensure high precision. You can seize the opportunity of a more efficient workflow without having to change your traditional impression process. How is this possible? Let us explain the easi team around cara Scan 4.0i.



easi team

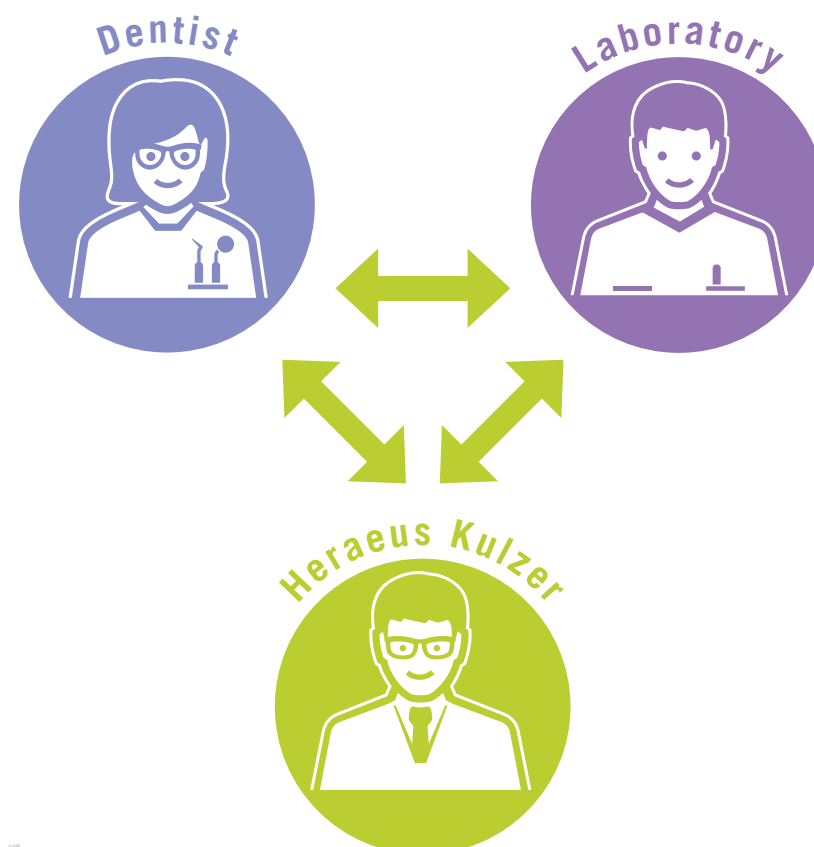

## eextraoral scanner

cara Scan 4.0i is an extraoral scanner that implements impression scanning and automated processing in your dental office. It is easy to use due to the convenient all-in-one system and simplified scanning workflow. Place the traditionally taken impression – triple tray, single or double full arch tray – into the scanner. The unit digitizes the impression and generates a detailed 3D file format. You and your patients benefit from the efficient system and workflow. Patients will be very pleased to get their restorations faster, as transportation of the impressions as well as fabrication of gypsum models are not mandatory.



## aautomated processing


With cara Scan 4.0i we are opening a door to the efficiency of digital data without time-consuming training and the complexity of intraoral scanning.

- cara Scan 4.0i heightens the precision of your indirect restorations in comparison with the model scan.
- It offers the opportunity to enlarge your base of digital documentation.
- In addition, it enables real-time collaboration with your dental technician. Due to the enhanced communication and shortened coordination processes, collaboration between lab and clinician becomes much easier.
- Thanks to the pre-installed teamviewer, your laboratory partner can promptly comment on critical impression situations, should the need arise.

## Sservice from Kulzer

cara Scan 4.0i comes with **Kulzer Koncierge**. That means we put all our know-how, digital expertise and long-standing experience with indirect restorations into

- superior customer service
- installation of scanner by a Kulzer technical consultant
- training and support
- consultancy with your dental technician partner and Kulzer Technical Service.

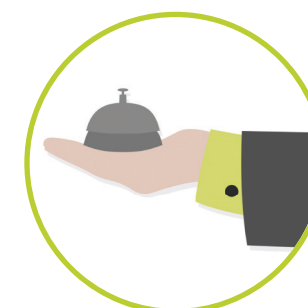

## iimpression material (scannable)

The workflow behind cara Scan 4.0i combines a scannable impression material, an extraoral scanner and an open data exchange platform. Our A-silicone **Flexitime Fast & Scan** provides higher precision and detail thanks to direct scanning without the need of powder.

As a state-of-the-art scannable impression material **Flexitime Fast & Scan** is the ideal supplement to cara Scan 4.0i.

